




CONTENTS: TRAY CS-OCC-20
(20 X PRO SENSORS, 2 LIP HOOKS)

USER INSTRUCTIONS

STERILE EO

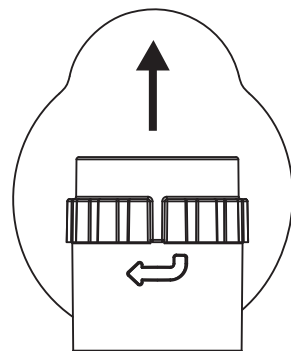


See CarieScan PRO
User Manual


Single Use Only


Do not use if
package is
damaged

1. THE COLLAR SHOULD BE PUSHED ONTO THE PRO (LOCATES WITH A 'CLICK') AND A SUITABLE CROSS INFECTION BARRIER EMPLOYED (SEE RECOMMENDATION IN PRODUCT IN PRO MANUAL AND ON OUR WEBSITE).
2. LIFT THIS INSTRUCTION CARD TO REVEAL THE PEELABLE SENSOR PACKAGING
3. PEEL BACK A TAB TO OPEN A SENSOR COMPARTMENT, REVEALING THE SENSOR TIP
4. WITH THIS SENSOR PACKAGE FACING TOWARDS YOU POSITION THE PRO WITH THE LCD FACING AWAY FROM YOU AND THE COLLAR POINTING DOWNWARDS.
5. PUSH THE PRO ONTO THE EXPOSED SENSOR UNTIL YOU HEAR A CLICK.
6. THE SENSOR IS NOW CONNECTED TO THE PRO.
7. PULL THE ASSEMBLY OUT OF THE PACKAGING AND USE IN ACCORDANCE WITH THE PRO USER MANUAL.
8. AFTER USE, THE SENSOR SHOULD BE REMOVED FROM THE PRO:
 - WITH THE LCD FACING TOWARDS YOU, PULL BACK THE COLLAR AND TWIST IT IN THE DIRECTION OF THE ARROW ON THE COLLAR.
 - THIS WILL DETACH THE SENSOR
 - THE SENSOR CAN BE DISPOSED OF STRAIGHT INTO YOUR CONTAMINATED OR SHARPS WASTE WITHOUT TOUCHING THE SENSOR.
 - CLEAN THE PRO, COLLAR AND CABLE WITH ALCOHOL BASED WIPES READY FOR THE NEXT USE

CarieScan Ltd, Technology Park, Dundee, DD2 1SW, UK
www.cariescan.com


0120

LT096